

t:connect[®]

Diabetes Management Application


Getting Started Guide

Congratulations on the purchase of your new insulin pump from Tandem Diabetes Care[®]. Your decision to use insulin pump therapy is a sign of your commitment to actively manage your diabetes.

This guide provides the basic steps to get you started using the t:connect[®] Diabetes Management Application and is not meant to replace the t:connect Application User Guide.

For more comprehensive information, please refer to the t:connect Application User Guide available online at:
<http://tconnect.tandemdiabetes.com/help/>

Need help? We are here for you 24 hours a day,
7 days a week at **1-877-801-6901**.

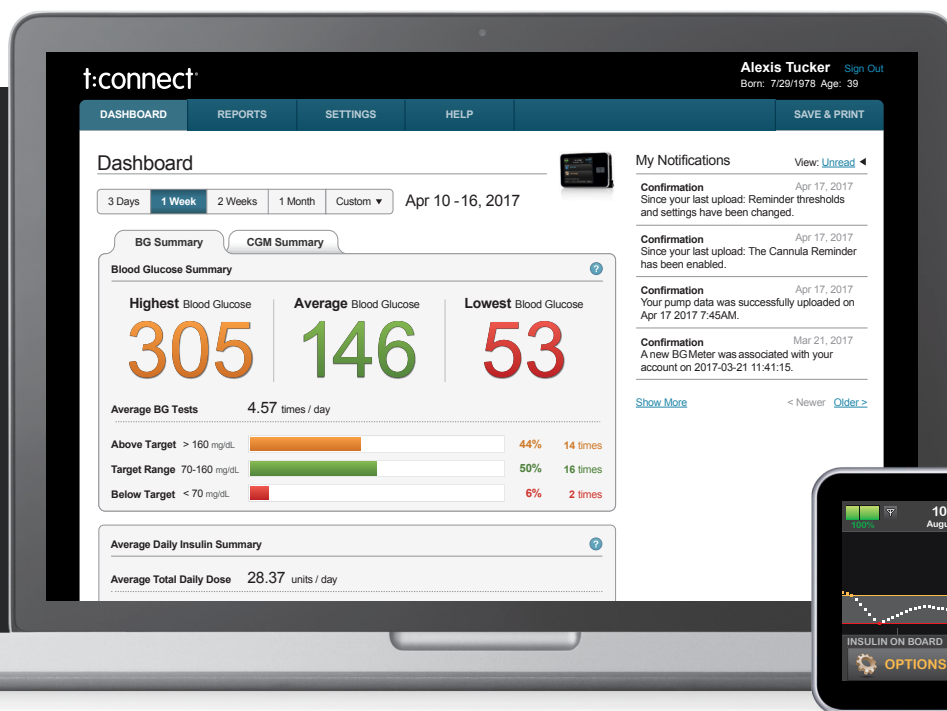

Three Simple Steps:


- 1 Install Software & Drivers
- 2 Create Account
- 3 Add Meter



Rx Only

What You Need:

1

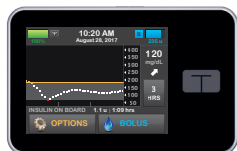

Your Pump

2



USB Cable

3



Computer

The computer will need internet access,
compatible operating system and
compatible internet browser

Operating System Requirements:

Microsoft®	Microsoft® Windows® 8.1	(64 bit only)
	Microsoft® Windows® 7	(32 or 64 bit)
Apple®	Mac® OS X® Yosemite	(v10.10.x)
	Mac® OS X® Mavericks	(v10.9.x)

Compatible Internet Browsers:

Internet Explorer®	Chrome™	Mozilla Firefox®	Safari®
10 or later (Windows Only)	63 or later (Windows and Mac)	52 or later (Windows and Mac)	9 or later (Mac only)

Intended Use

The t:connect® Diabetes Management Application is intended for use by individuals with diabetes mellitus who use Tandem Diabetes Care® insulin pumps, their care givers, and their healthcare providers in home and clinical settings. The t:connect Application supports diabetes management through the display and analysis of information downloaded from Tandem Diabetes Care insulin pumps and specified blood glucose meters.

⚠ Warnings

United States (U.S.) Federal law restricts this device to sale by or on the order of a physician. Consult with your healthcare provider before making any changes to your treatment. The t:connect Application is intended for use by patients and healthcare providers who are familiar with the management of diabetes.

Important User Information

Thoroughly review all product instructions before using the t:connect Application for warnings and other important user information. Discuss with your healthcare provider how you should use data from the t:connect Application to help manage your diabetes. Incorrect use might lead you to misunderstand the information provided by the t:connect Application.

Need help? We are here for you
24 hours a day, 7 days a week.

(877) 801-6901



TANDEM®
DIABETES CARE

11075 Roselle Street • San Diego, California 92121

www.tandemdiabetes.com

© 2018 Tandem Diabetes Care, Inc. All rights reserved. t:connect and Tandem Diabetes Care are registered trademarks of Tandem Diabetes Care, Inc. All other trademarks are property of their respective owners. For a list of patents, please see www.tandemdiabetes.com/patents.


007835_B



1

Install the Uploader Software and Drivers

- 1) Go to <http://tconnect.tandemdiabetes.com>. Click **Get Started** to download the t:connect Uploader.

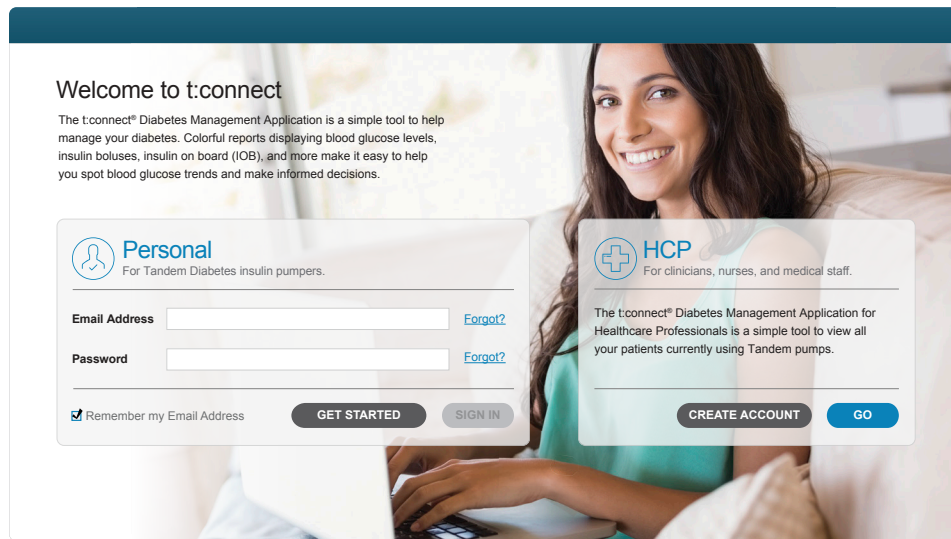

- 2) Click on the green button to download the t:connect Uploader, and follow the online steps to install it on your computer.



Note: The software download experience may vary depending on the web browser you are using.

2

Create an Account and Upload Your Pump

- 1) Run the t:connect Uploader software.

The t:connect Uploader software should automatically start after you complete the installation. If it does not automatically start, double click the program's icon on your desktop or applications folder.

- 2) Connect your pump to your computer.



Connect one end of the USB cable to your pump and the other end to a USB port on your computer.

Note: You do not need to disconnect your infusion set or stop insulin delivery.

- 3) The t:connect Uploader will detect your pump and display its serial number on the computer screen. Click the **Start Upload** button.

Start Upload

- 4) The t:connect Uploader will prompt you to create a new account. Click the **Get Started** button.

Get Started...

- 5) Select account type.

- ☐ **Personal**
☐ **Parent, Guardian or Caretaker**

Note: If you already have an account, just sign in with your username (email address) and password to add your new pump to your existing t:connect account.

- 6) Fill in the necessary details to create your account.

My First Name	My Last Name
<input type="text"/>	<input type="text"/>
Email	Confirm Email
<input type="text"/>	<input type="text"/>
My Password (8 characters minimum)	Confirm Password
<input type="text"/>	<input type="text"/>

- 7) Enter your Blood Glucose Limit values to personalize and color code your reports.

● High Blood Glucose Limit (mg/dL)	<input type="text"/>
● Target Blood Glucose Range (mg/dL)	<input type="text"/> - <input type="text"/>
● Low Blood Glucose Limit (mg/dL)	<input type="text"/>

Congratulations!



You have created your t:connect account.

Check your email inbox for a welcome message from Tandem Diabetes Care, and click the link to confirm your email address.

3

Add Your Meter

What You Need:

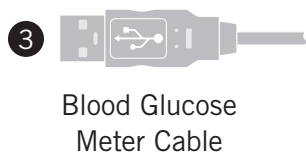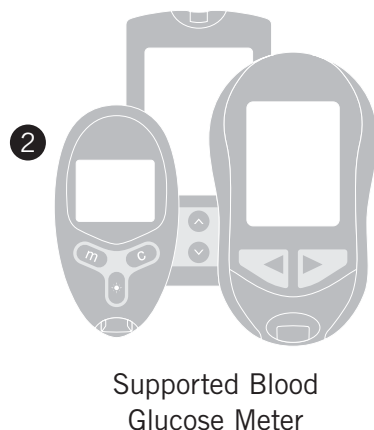

Supported BG Meters:

LifeScan®	OneTouch® Verio®IQ
	OneTouch® UltraMini®
	OneTouch® Ultra®2
Roche®	Accu-Chek® Aviva*
	Accu-Chek® Compact Plus*
Abbott Diabetes Care®	FreeStyle® Lite®
	FreeStyle® Freedom Lite®

* Accu-Chek® Aviva and Accu-Chek® Compact Plus are not currently compatible with Mac® OS X®.

- 1) Plug a supported BG meter in to your computer.



- 2) Run the t:connect Uploader.
Click the **Start Upload** button.

Start Upload

- 3) The t:connect Uploader will identify your new BG meter. Click the **Link Meter** button.

Link Meter...

- 4) Sign in to your t:connect account to link your BG meter.

Email Address

Password

☐ Remember my email address.

Sign In 